

## INSTRUCTIONS FOR USE OF KIP

KIP is to be treated as though it were a living patient with respect to injection techniques. Instructions for the various injection processes are in the province of physicians, rather than of the manufacturer of KIP. Instructions on KIP are concerned with its differences in use as compared with patients. The following differences should be noted:

- 1) After insertion of the syringe, no liquid is discharged, as would be the case with a patient;
- 2) Positioning for the injection is done by moving KIP as a single unit, rather than by movement of one part of the body with respect to another. Positioning can be facilitated by use of pillows to establish a stable position for each procedure;
- 3) In the unlikely event of damage to soft tissue injection sites, both the latex skin and the underlying gels can be replaced. To order replacements, specify the location on the body, and whether the replacement is of the skin, or the underlying soft tissue, or both;
- 4) KIP may develop an objectionable odor if stored in a closet. When so stored, a plastic garbage bag should be drawn over KIP (but injections must be made with the bag removed).